


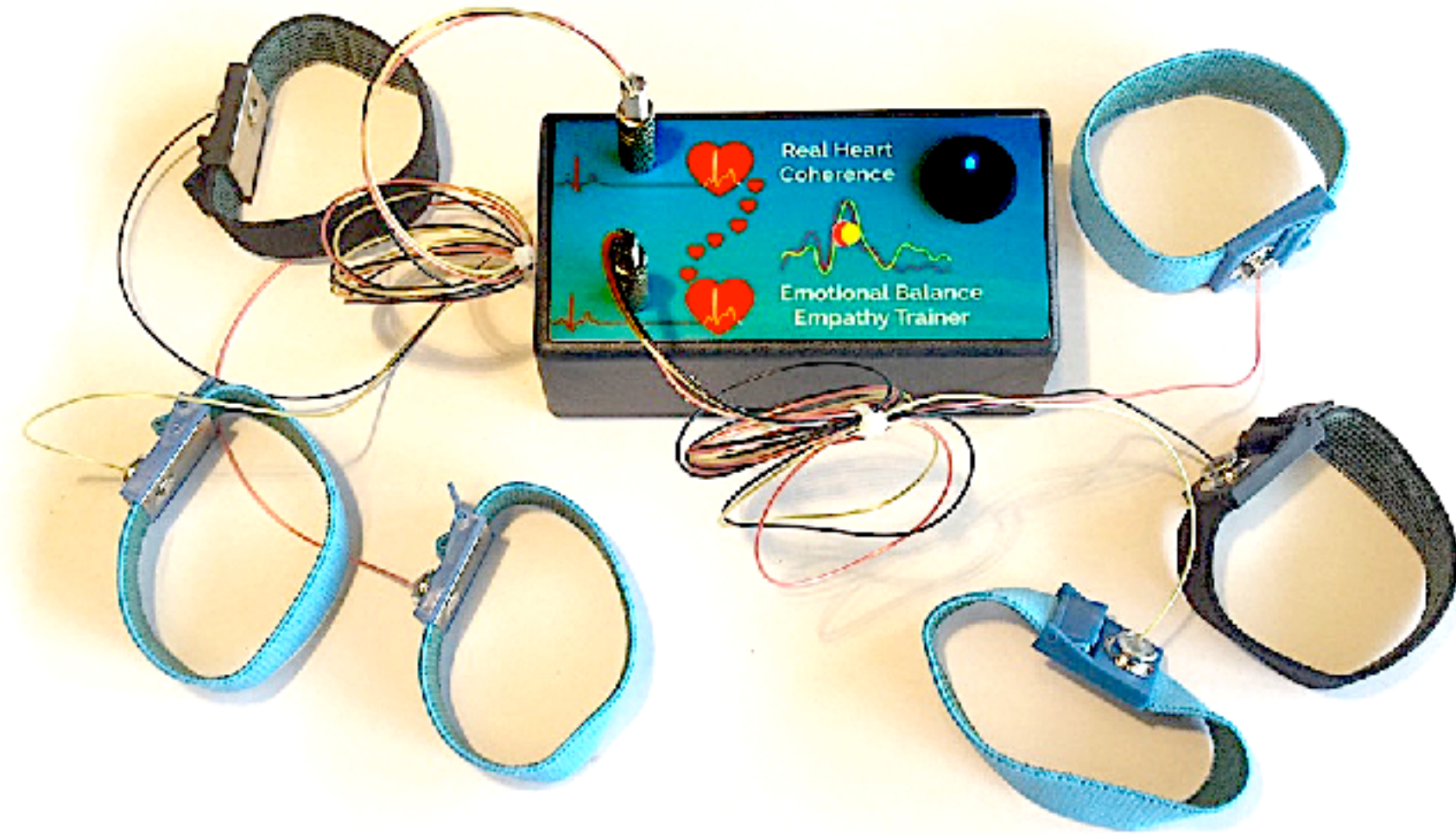
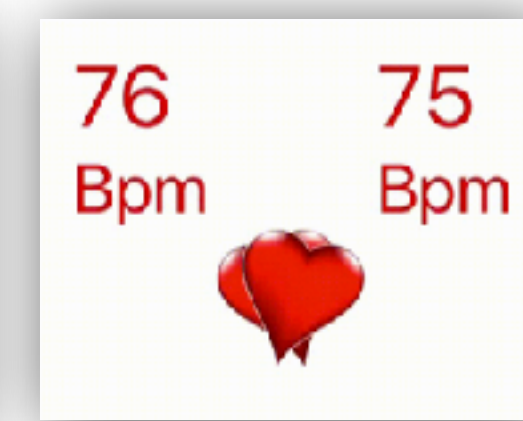
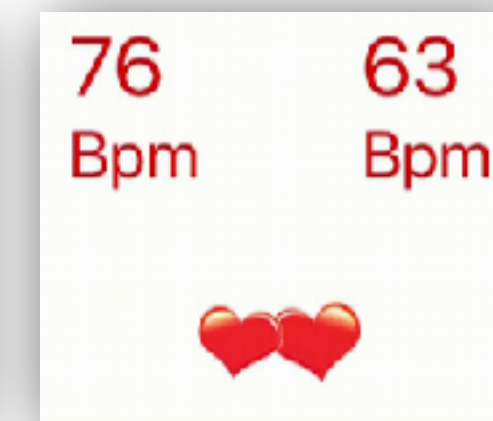
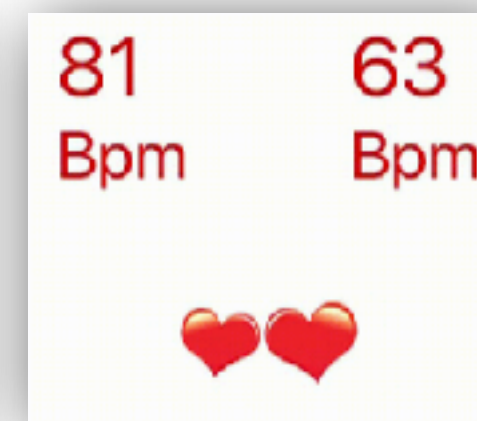


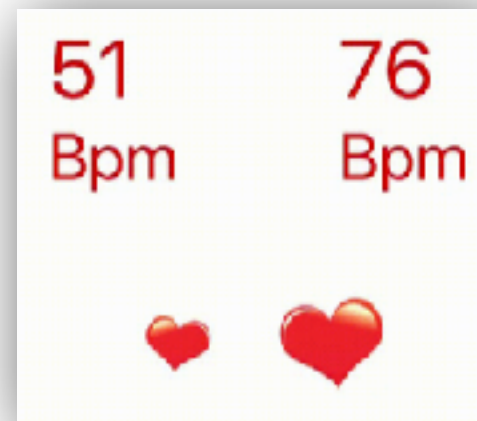

Real Heart Coherence Emotional Balance

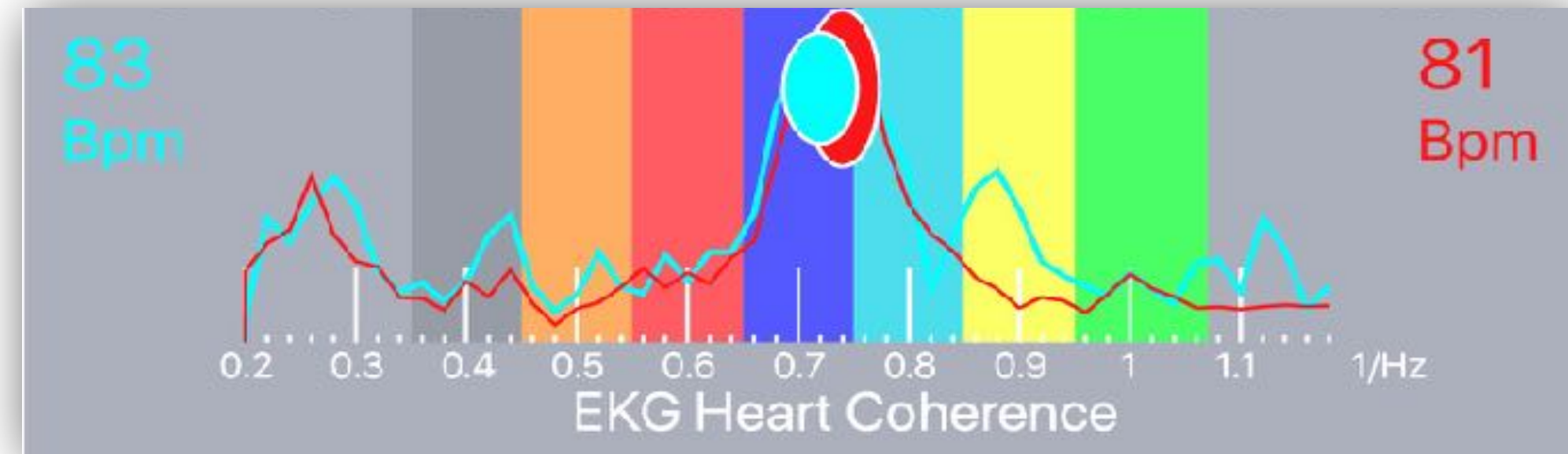

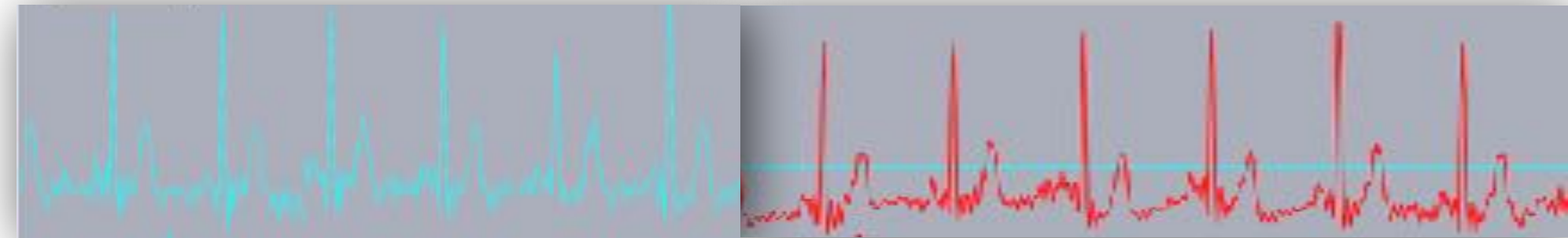



Power light

Before first use
!!! Make sure the device is fully charged !!!
Switch OFF the power light
Connect USB plugs to the device and to a USB charger
While charging, the red light is on
When fully charged, the red light turns off




Charging light

RealCoherence amplifier
comes with
2 sets of wrists & 2 sets of stickers

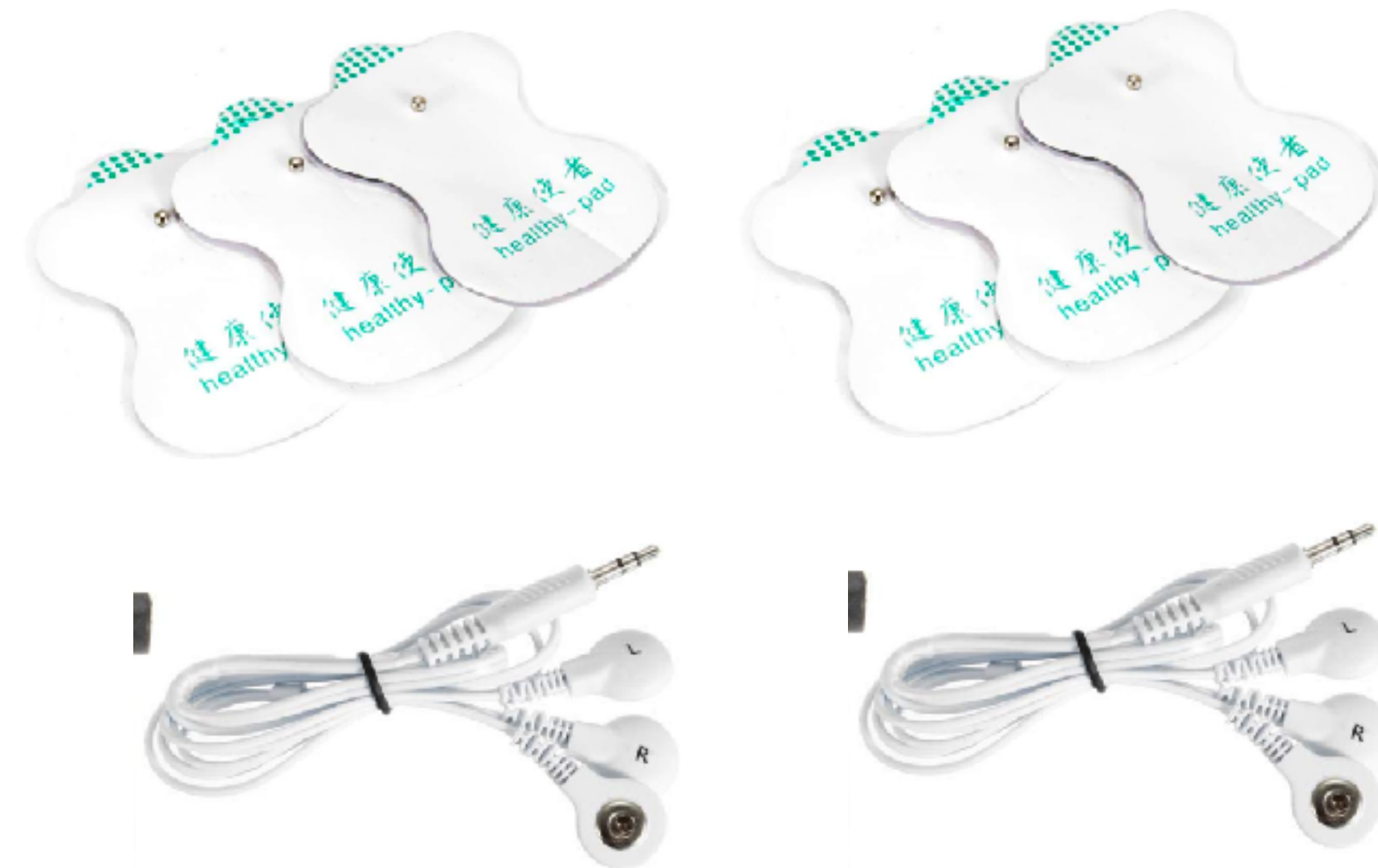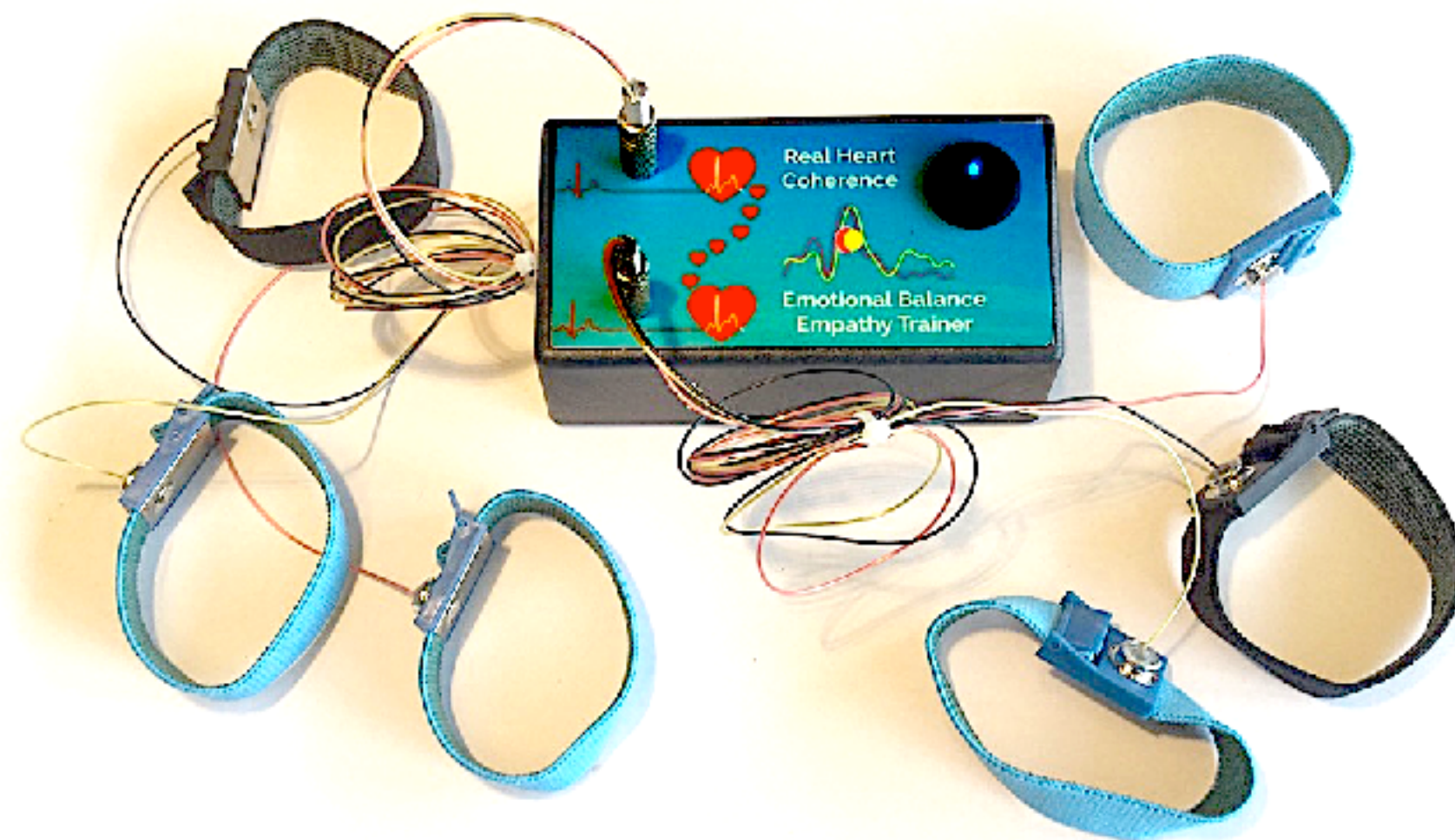


Stikers positions on the body

Stikers do not need contact gel

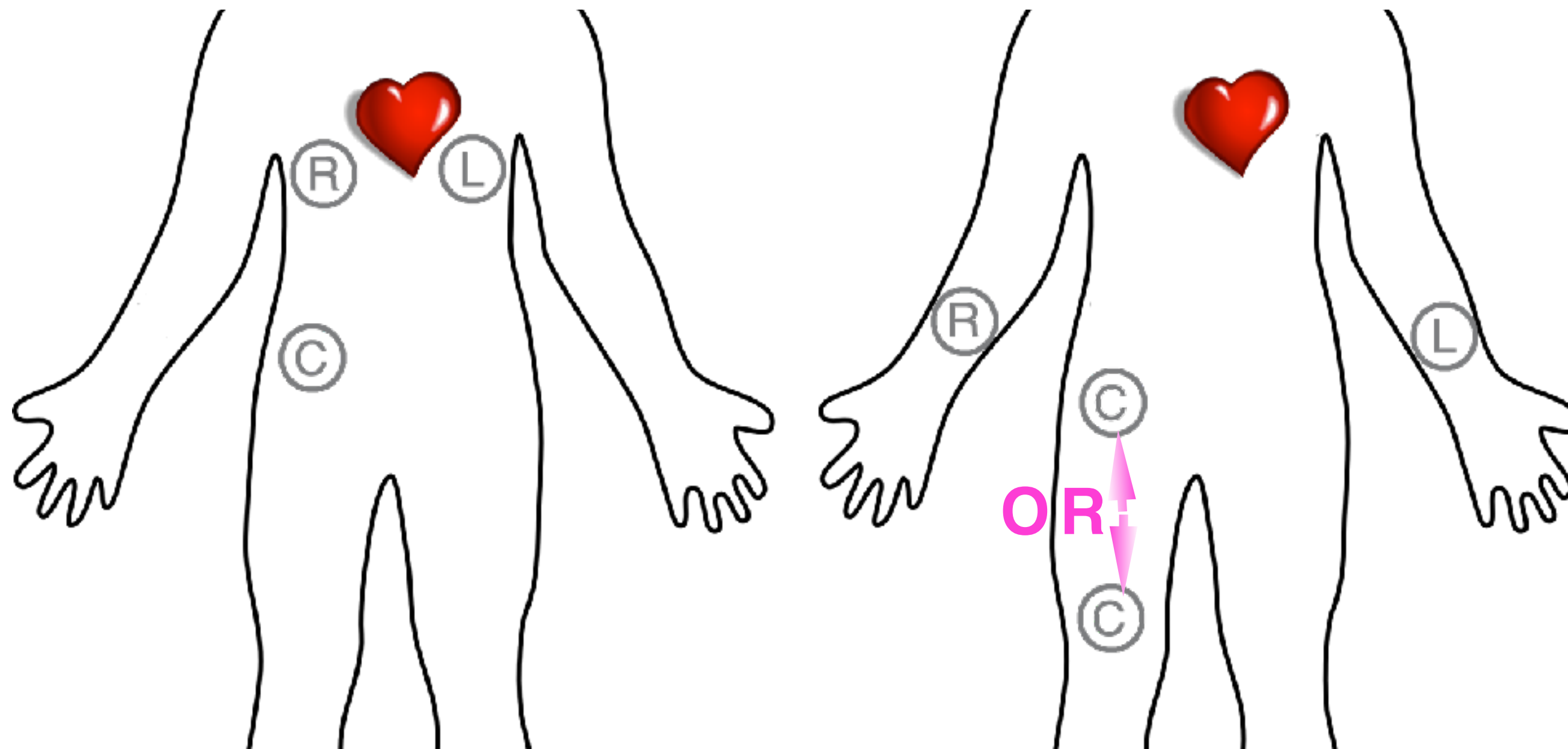

To work properly,
Wrists need contact gel


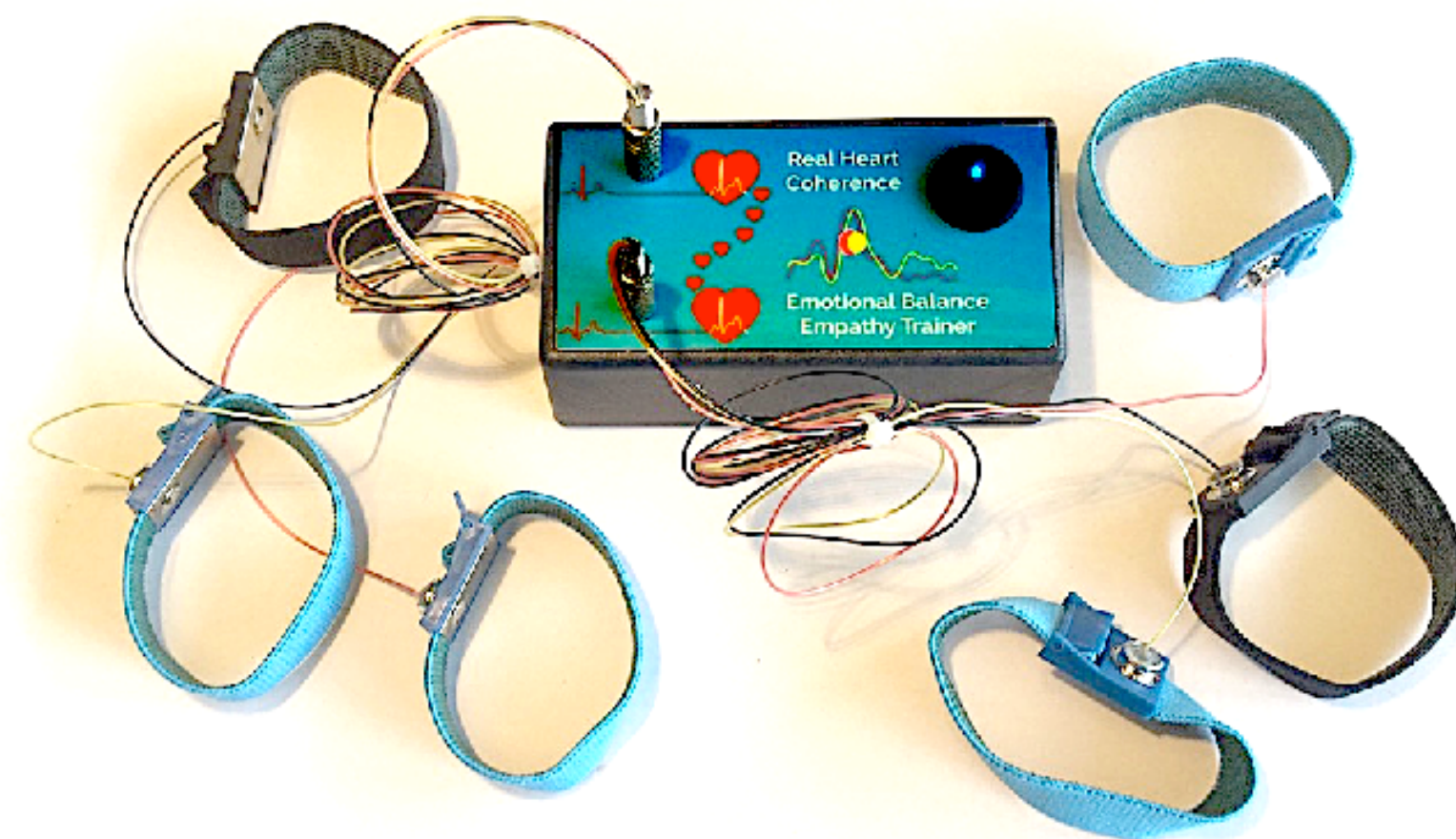


Place a drop of gel
between
the inner metal part of the wrist and your skin
A small bag containing some gel is provided in
the box

To order more contact gel, be sure to select
electrode gel like for example:


https://www.amazon.com/Spectra-PAR12-02-Parker-Laboratories-Electrode/dp/B00AMGUZ70/ref=sxbs_sxwds-stvp?keywords=contact+gel+electrode&pd_rd_i=B00AMGUZ70&pd_rd_r=44cdd30f-0a06-4881-9966-3562d82ee27f&pd_rd_w=4DA1I&pd_rd_wg=UxE7M&pf_rd_p=a6d018ad-f20b-46c9-8920-433972c7d9b7&pf_rd_r=B355T90GV9S2CP5G4D3W&qid=1560871041&s=gateway

How to connect wrists - 2 Users

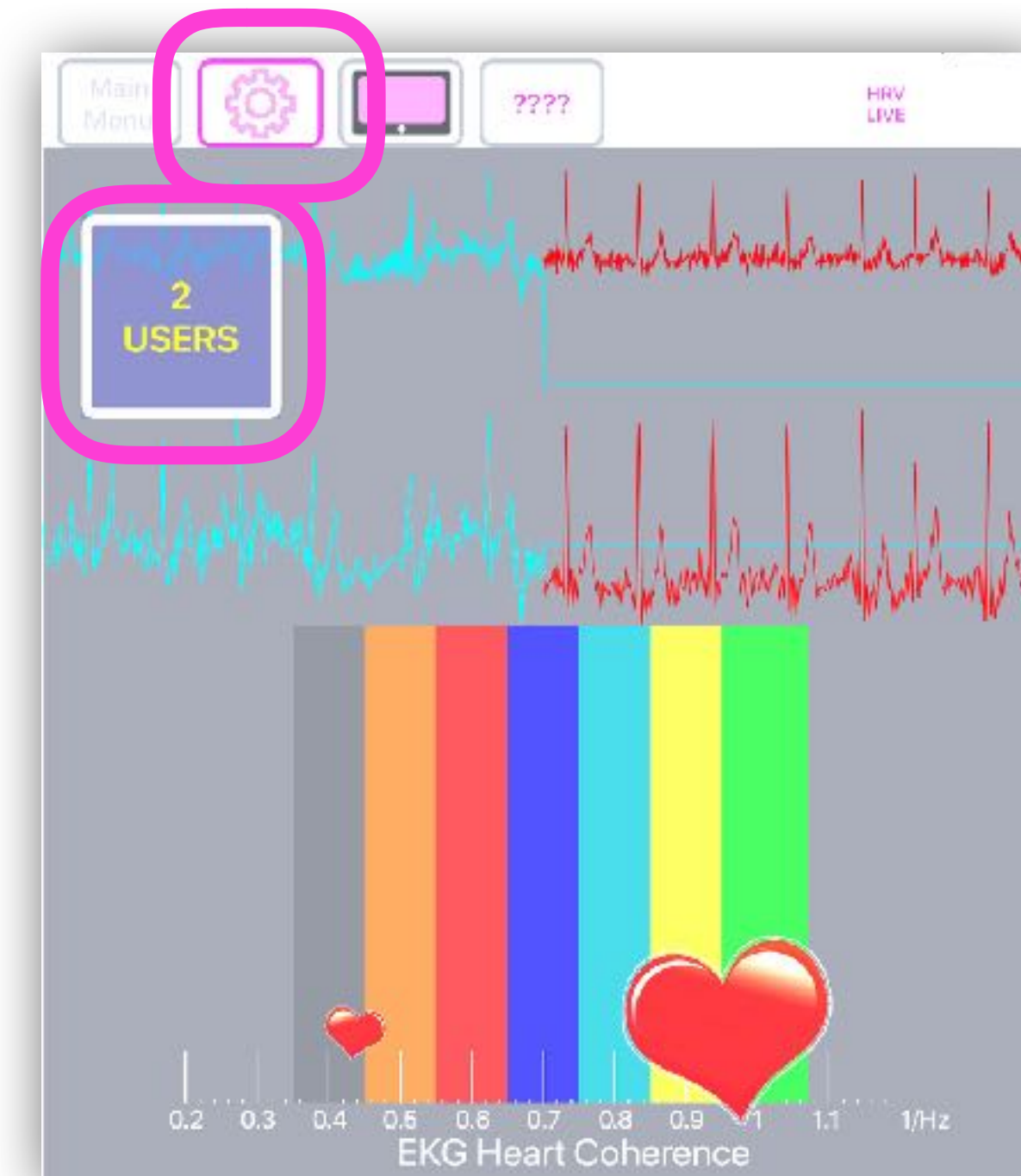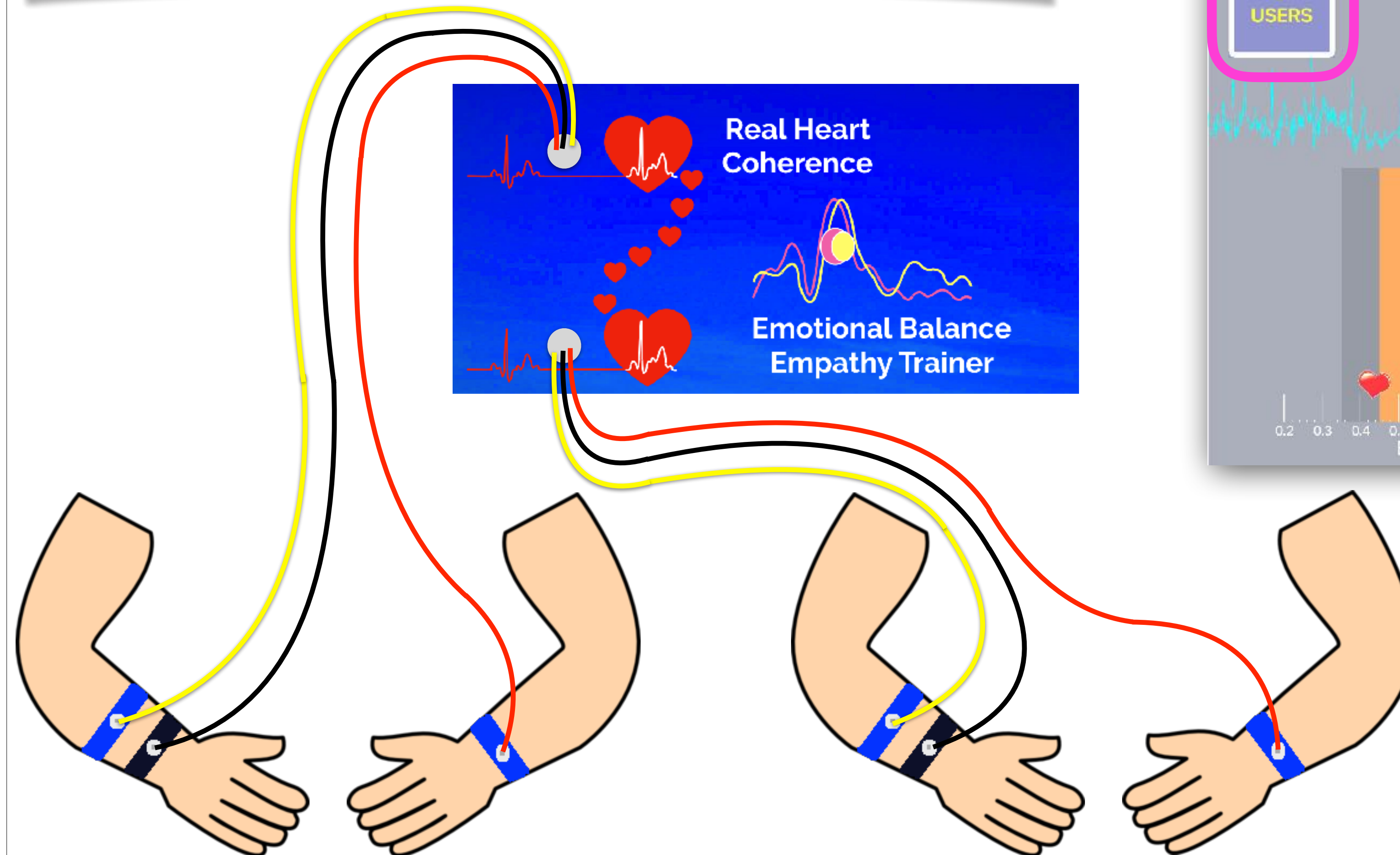

How to connect wrists Single User

Bottom
plug

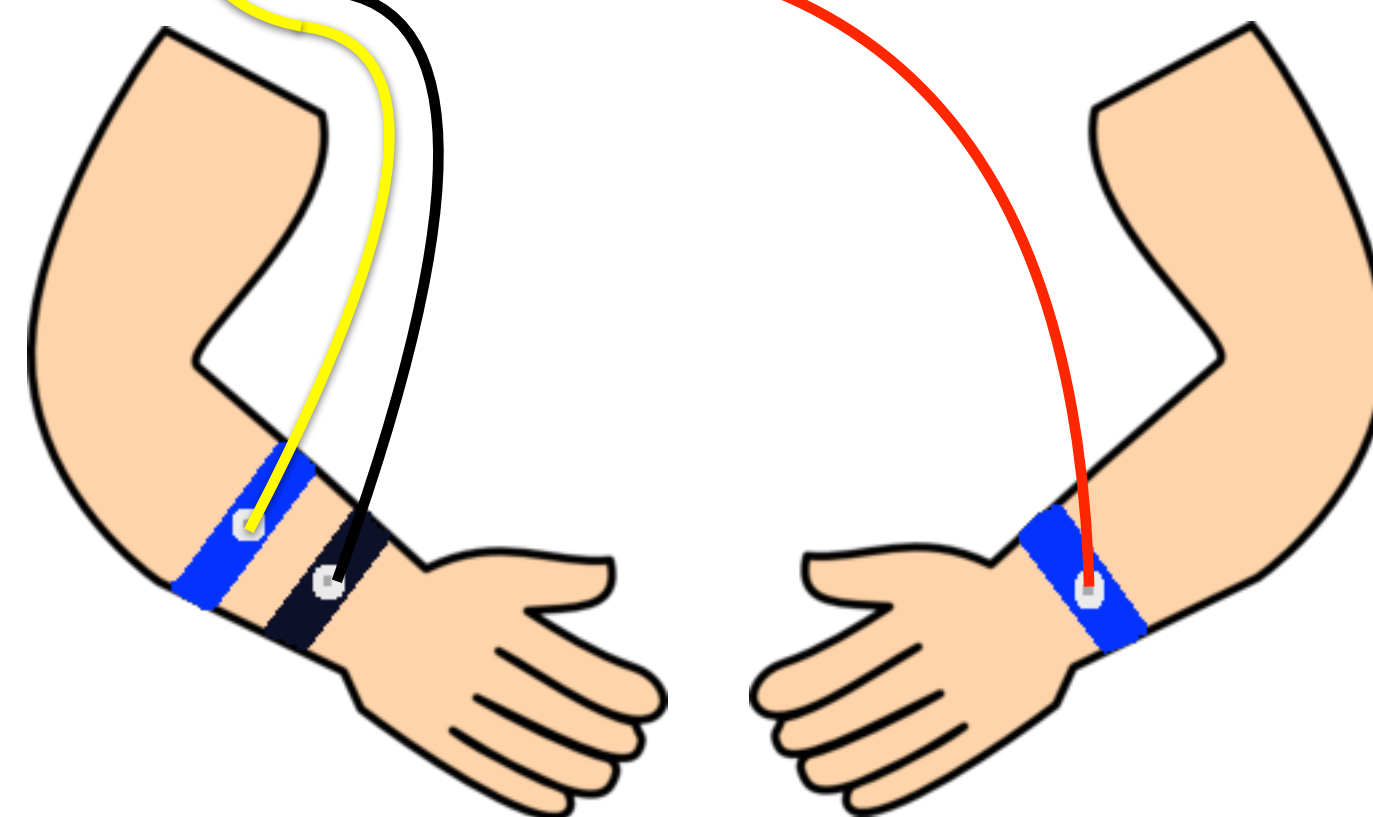

For FlameinMind App Users




Real Heart Coherence - Emotional Balance

is only working in FlameinSound App.
As FlameinSound App is included in FlameinMind App,
select FlameinSound button.
You will have to quit the app and start it again
to go back to FlameinMind App.




FlameinMind App

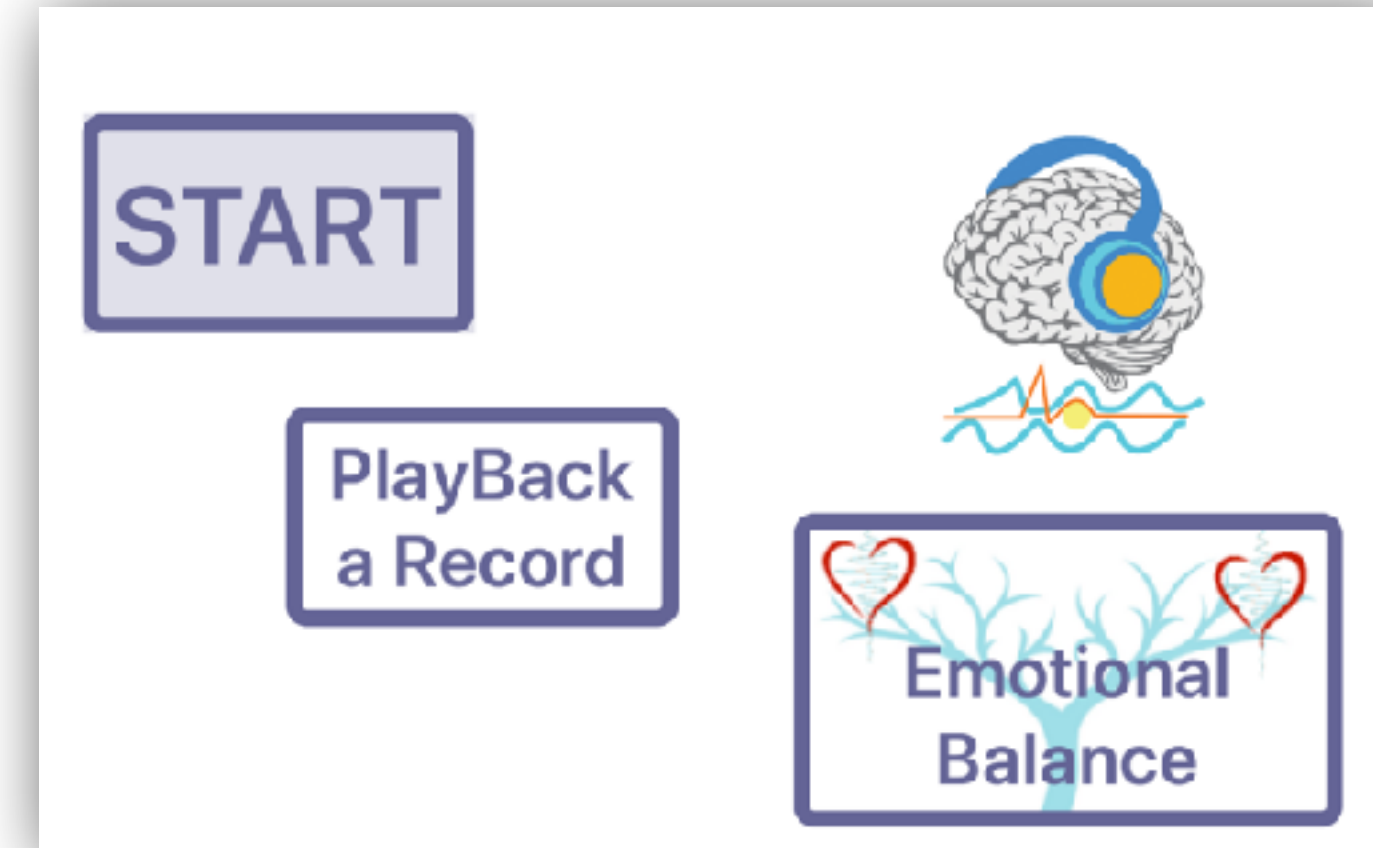

FlameinSound App

START

PlayBack
a Record

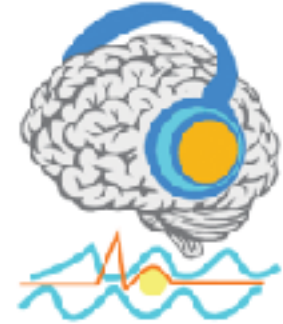

3 main parts in the App (selected with lower buttons):

Sound Wave Generator (lower right button)
Optional connection to HRV sensors (lower middle button)
HRV Waves Analyser (lower left button)

NO HRV SENSOR SELECTED

To play Binaural sounds, select lower right button.
To use HRV sensors, make sure BlueTooth is ON.
Use the lower middle button to connect a sensor.
Then touch the Analyser lower button to go to the Analyser.

Select Playback/live (if an HRV sensor is connected)
Then press START to start a wave session display

Flame In Sound 1.0 (5) <http://www.flameinmind.com/flameinsound>

Waves
ANALYSER

Sensors
Search

B.Natural
SOUNDS

In FlameInSound App

!!!! First !!!!

Touch the middle lower button
Search Sensors

to connect the
Real Heart Coherence
device

16:37 Mer, 19 juin

100 % 



Touch SCAN
Select **DSD TECH** or **BT05**
Pulses are displayed at the top of the screen

Touch SCAN to search for your Bluetooth sensor:
BERRY BM1000C - KYTO HRM 2935 - POLAR H7
or RealHeart Amplifier (DSD TECH or BT05)

Touch the device name to select it from the list.

Then touch the lower left button to go to the ANALYSER.

BlueTooth Hrv Sensors

Scan

disconnected

BlueTooth Hrv Sensors

Stop Scan

scanning...

iPhone de Patrick

DSD TECH

MacBook Pro de Patrick

BlueTooth Hrv Sensors

Stop Scan

scanning...

BT05

iPhone de Patrick

MacBook Pro de Patrick



Swipe Left/Right to change Time Base

Touch SCAN to search for your Bluetooth HRV sensor:
BERRY BM1000C - KYTO HRM 2935 - POLAR H7

Touch the device name to select it from the list.

Then touch the lower left button
to go back to the ANALYSER.

BlueTooth Hrv Sensors

Disconnect

connected

iPhone de Patrick


MacBook Pro de Patrick

BT05


Waves
ANALYSER

Sensors
Search

BiNatural
SOUNDS

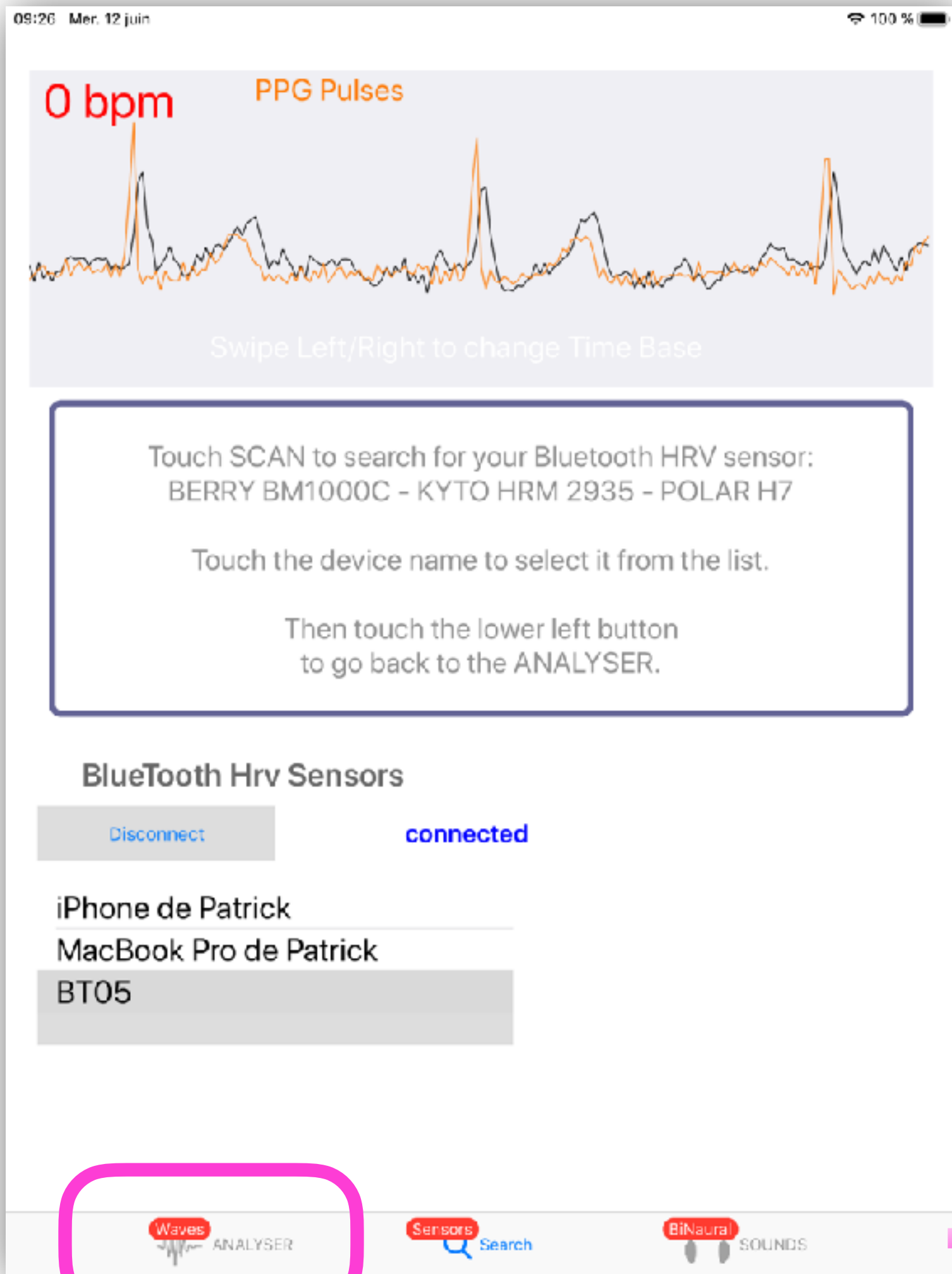

To Go to the Main Menu
Touch the lower left button

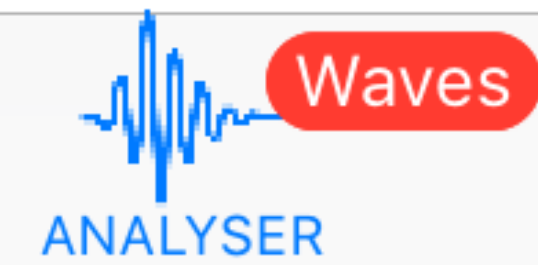

START

PlayBack a Record

Emotional Balance

3 main parts in the App (selected with lower buttons):

- Sound Wave Generator (lower right button)
- Optional connection to HRV sensors (lower middle button)
- HRV Waves Analyser (lower left button)


NO HRV SENSOR SELECTED


To play Binaural sounds, select lower right button.
To use HRV sensors, make sure BlueTooth is ON.
Use the lower middle button to connect a sensor.
Then touch the Analyser lower button to go to the Analyser.

Select Playback/live (if an HRV sensor is connected)
Then press START to start a wave session display

Flame In Sound 1.0 (5) <http://www.flameinmind.com/flameinsound>

Waves ANALYSER Sensors Search BiNaural SOUNDS

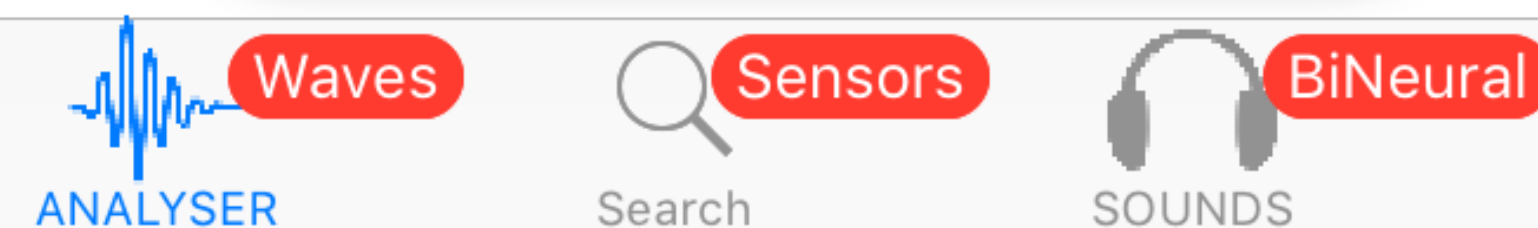


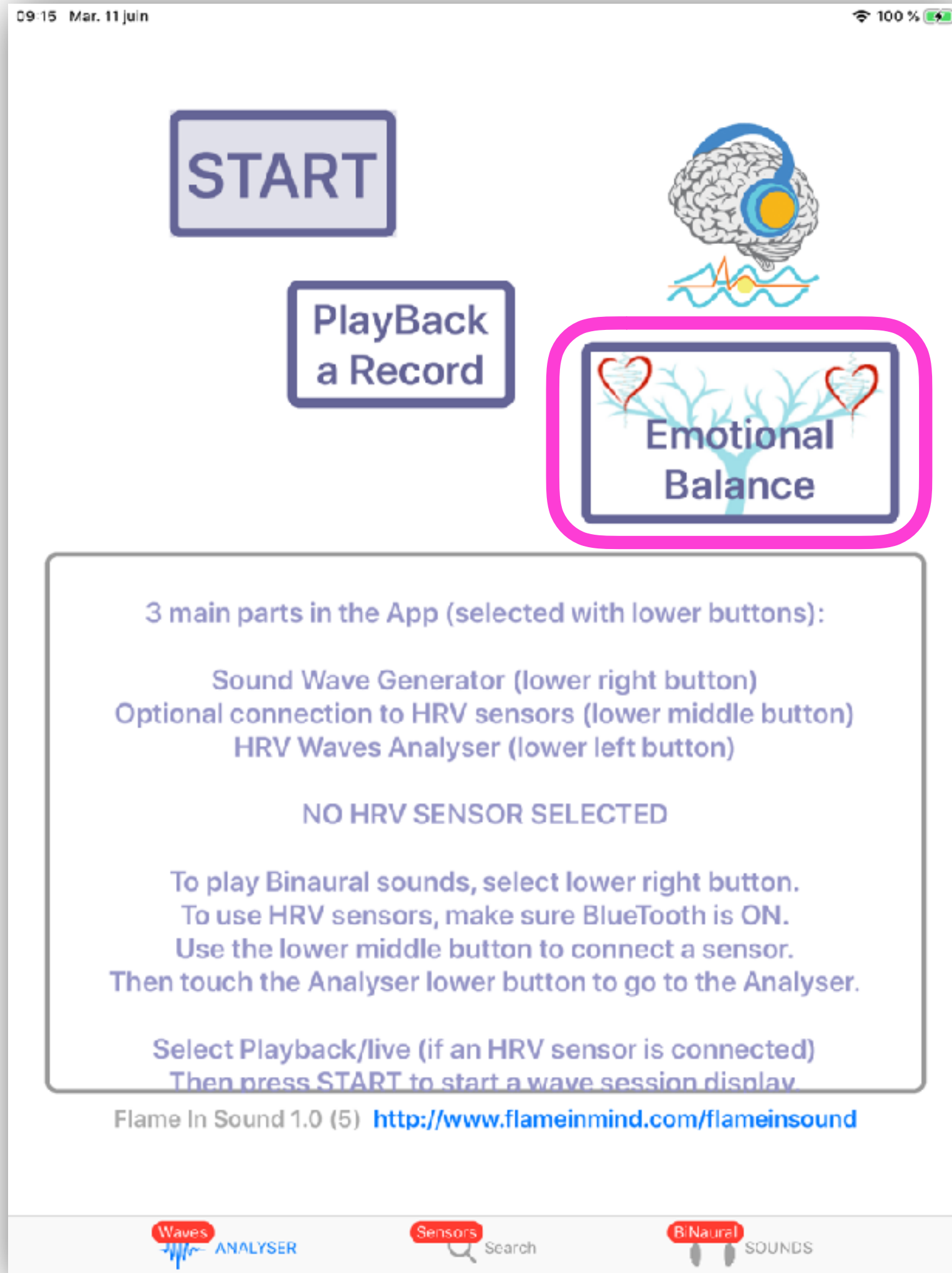

Select

***Emotional
Balance***

Select Single User - 2 Users

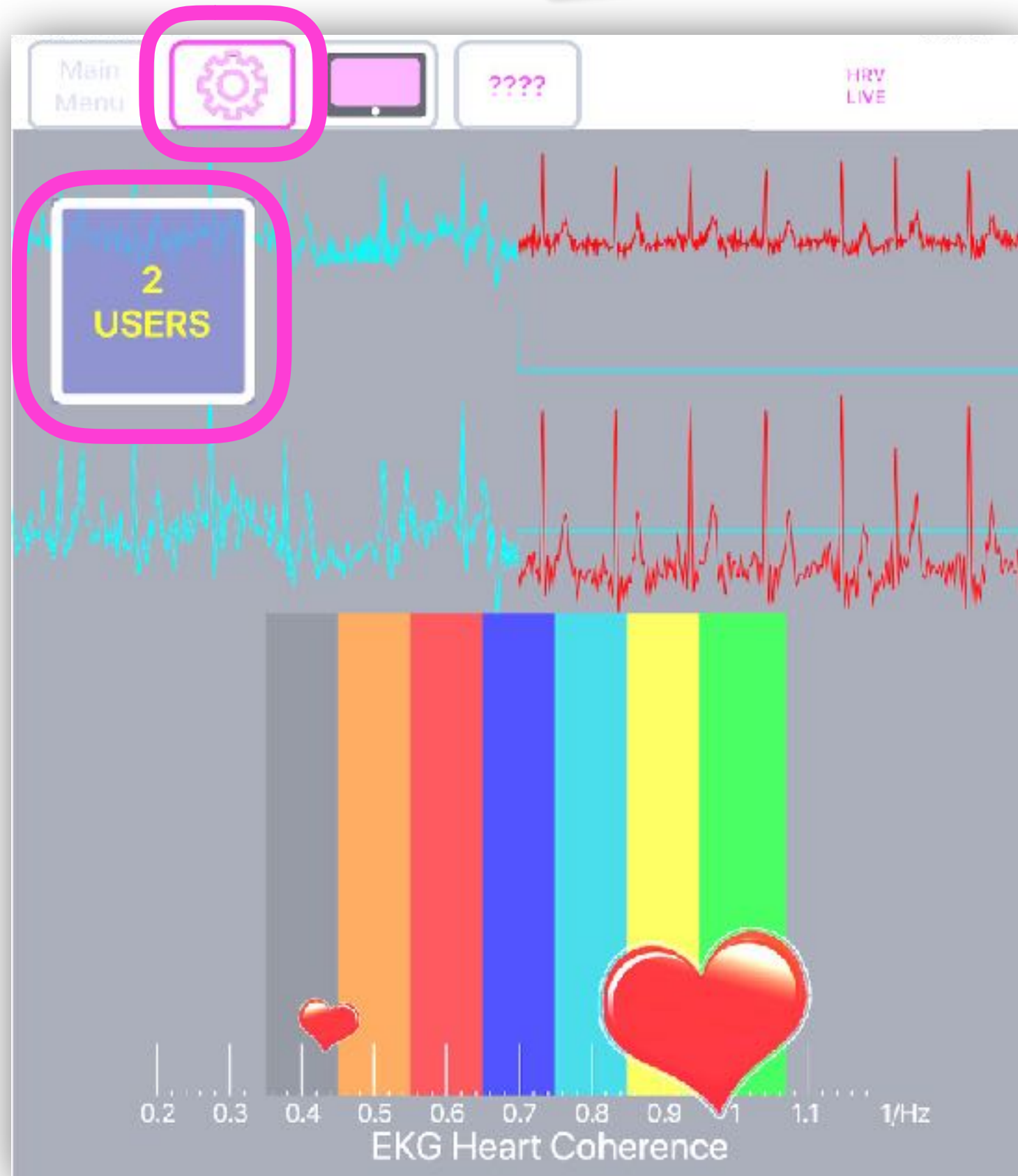

Display Options

17:26 Mar. 18 juin

100 % 

Main
Menu



????

HRV
LIVE

Display
Pulses

e->
motion
Graph

e->
motion
Spectrum

Pulses
Raw Data

FFT
Spectrum

HEARTS
Display


All display options can
be selected separately

Display Options

Single User example


e->
motion
Spectrum

The size of the
spectrum peak
reflects
internal coherence
in EKG

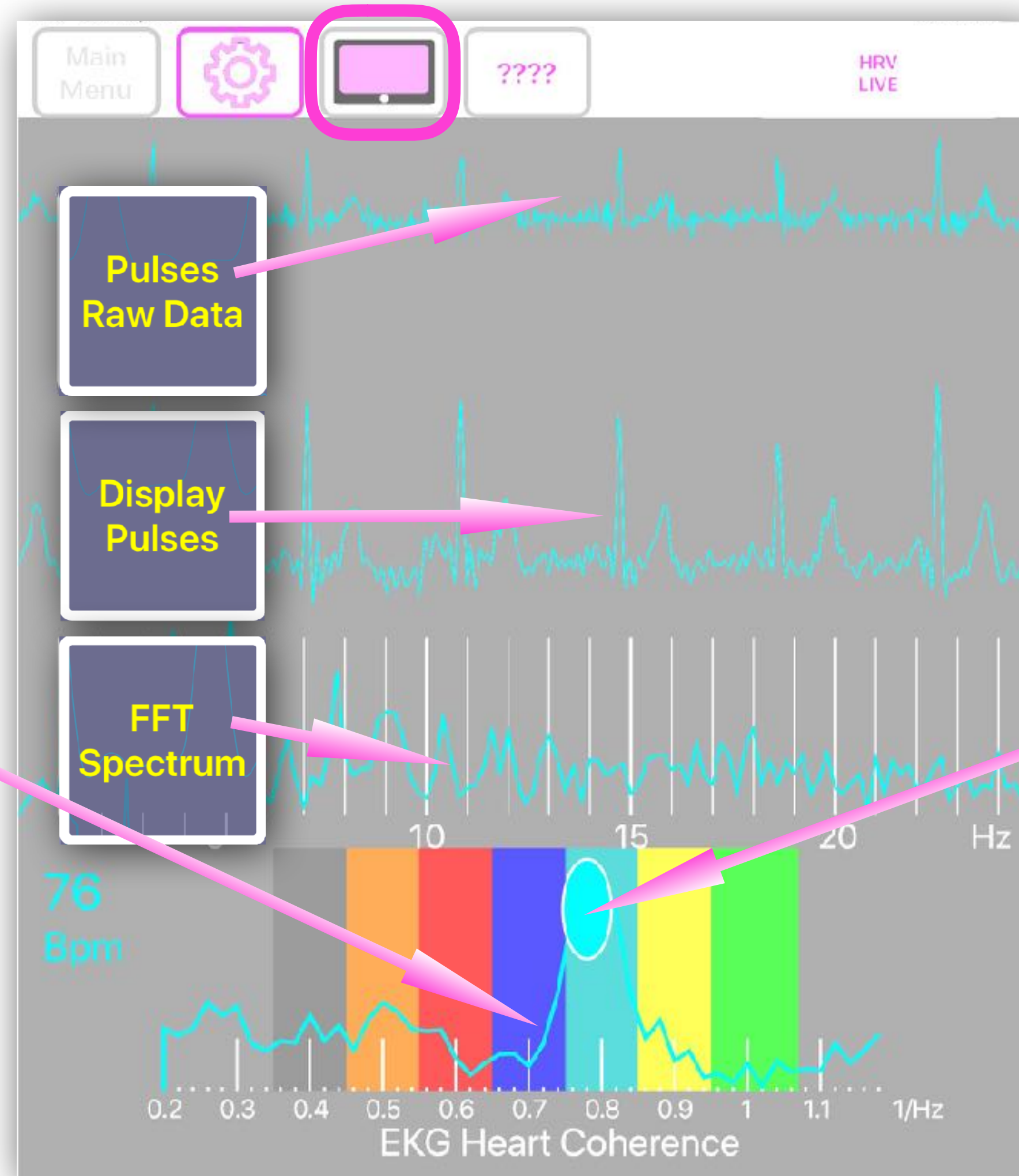

e->
motion
Graph

The size
of the bullet
reflects
internal coherence
in EKG


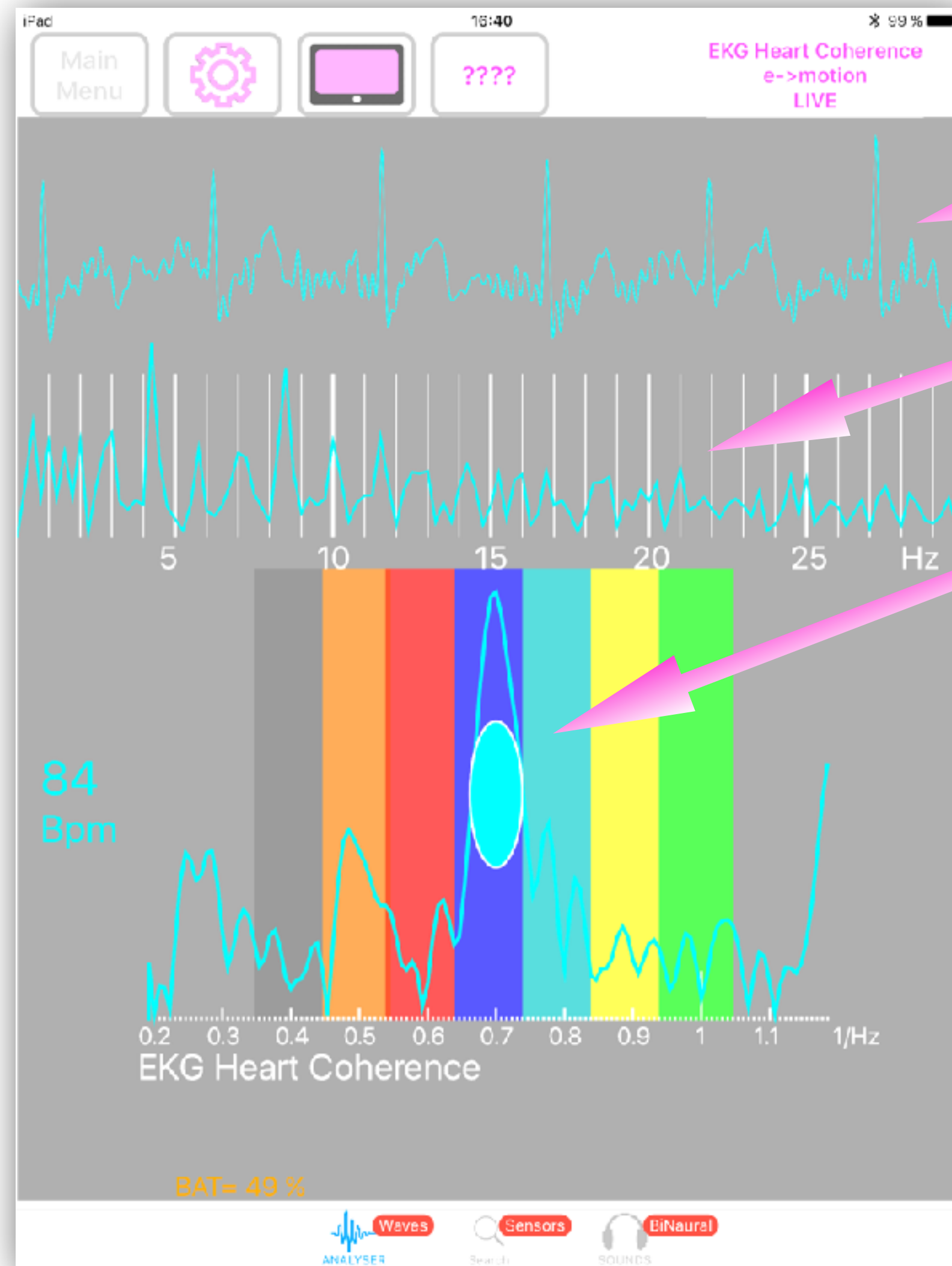



EKG Heart Pulses

EKG 1st order spectra





Internal Coherence in Heart EKG
EKG 2nd order spectra - CEPSTRUM

The size of the bullet increases
according to coherent state

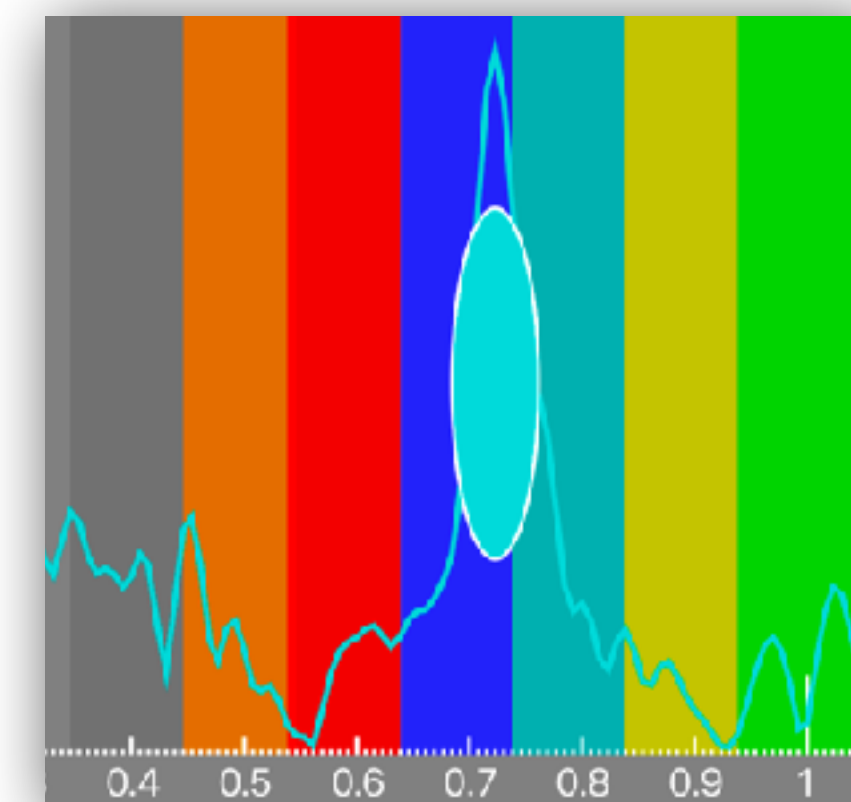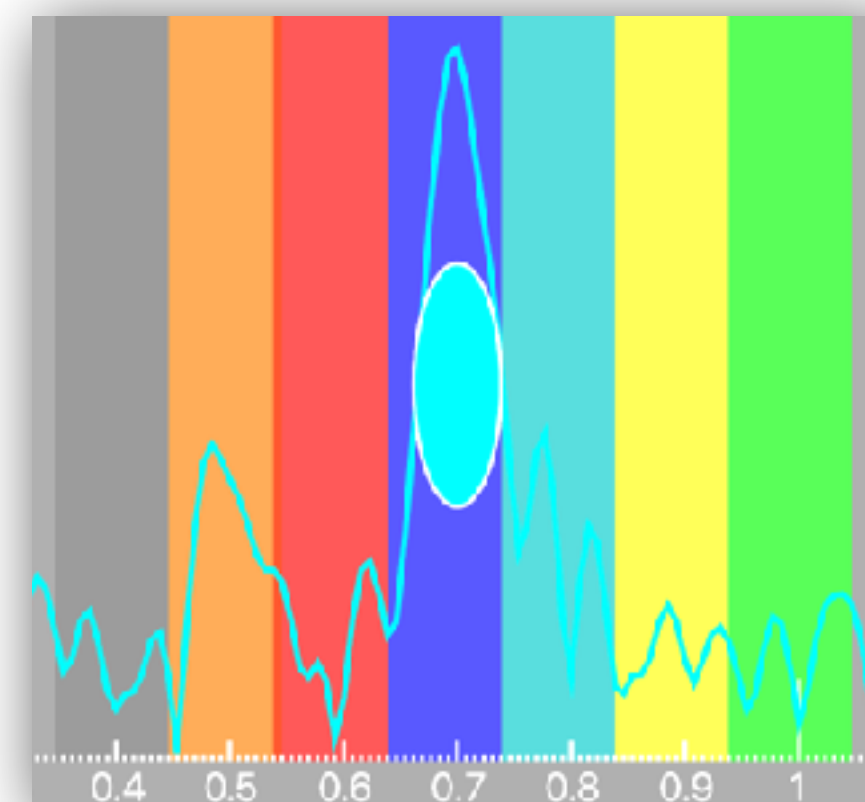

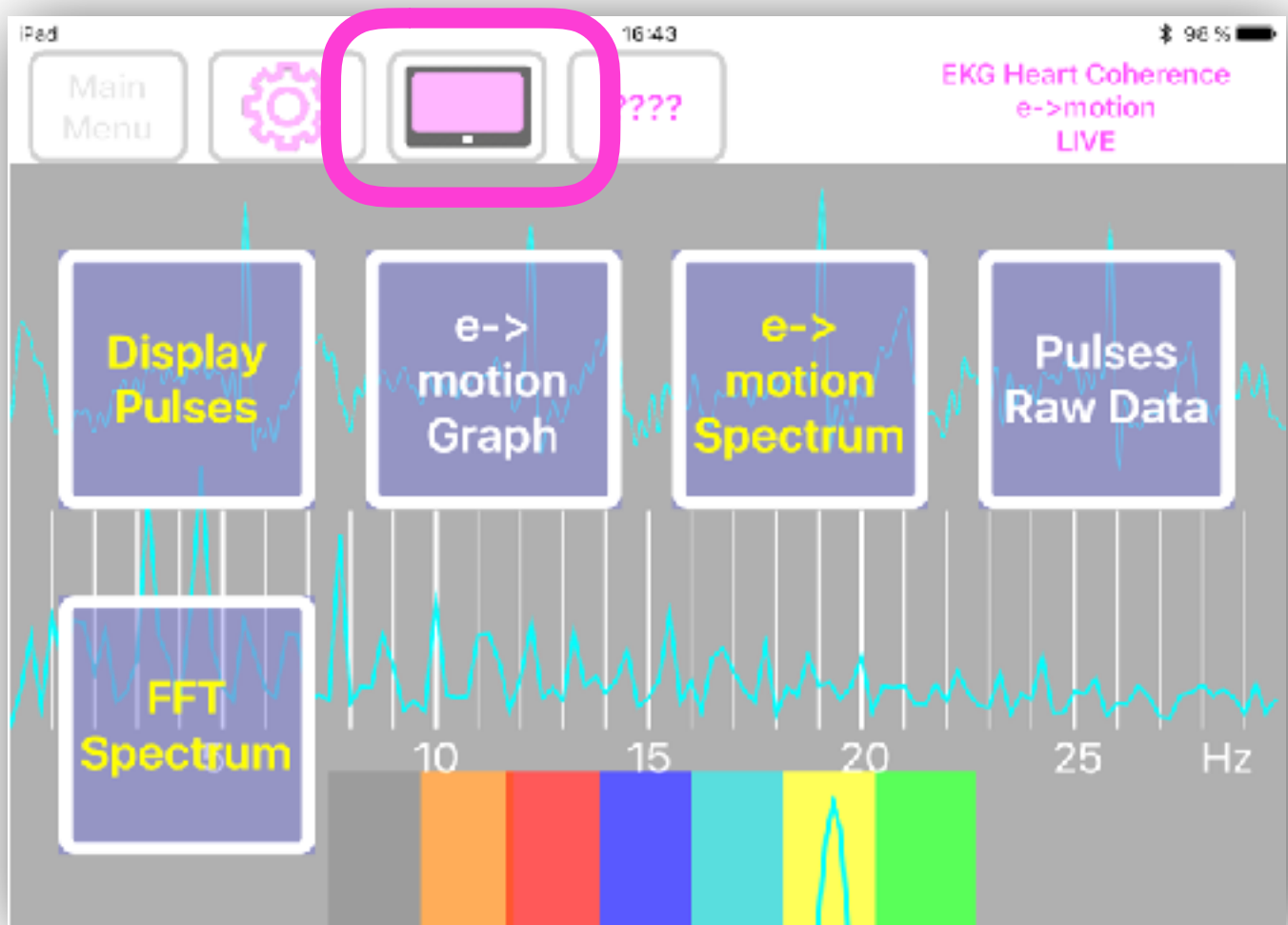

Different kinds of displays
According to Display Settings

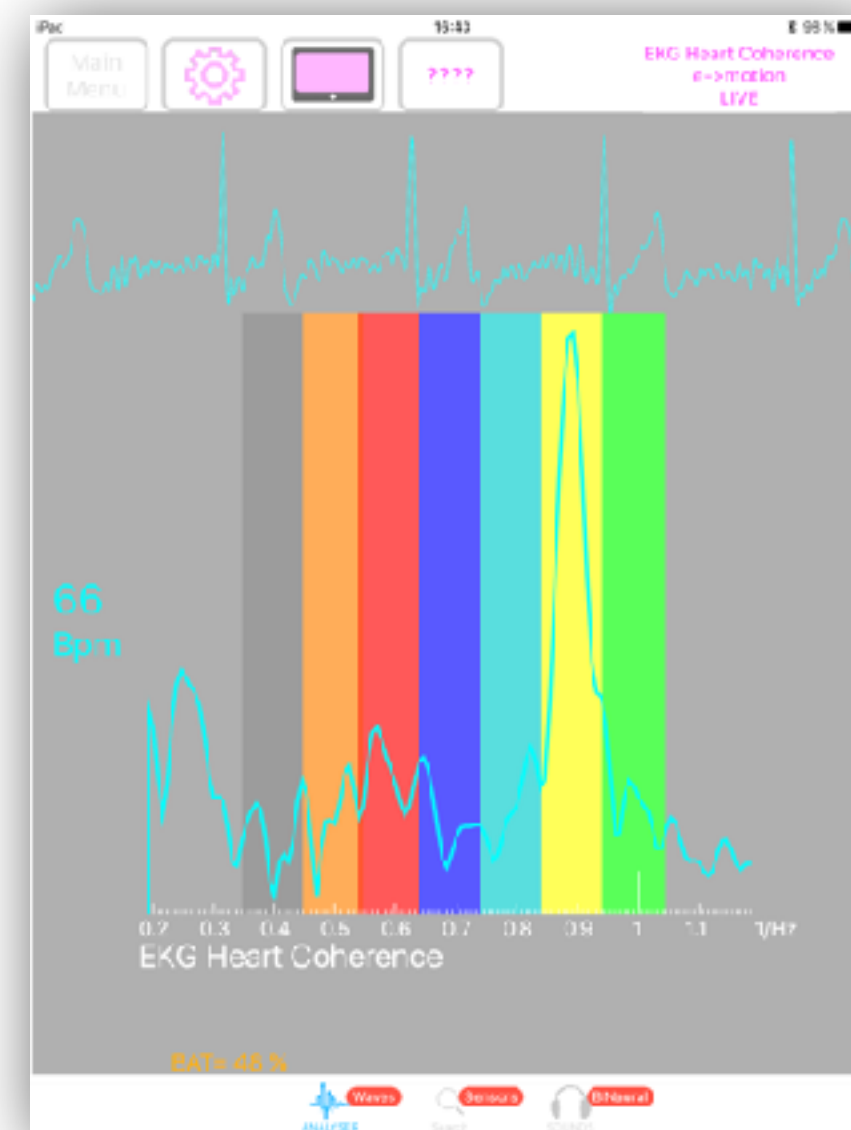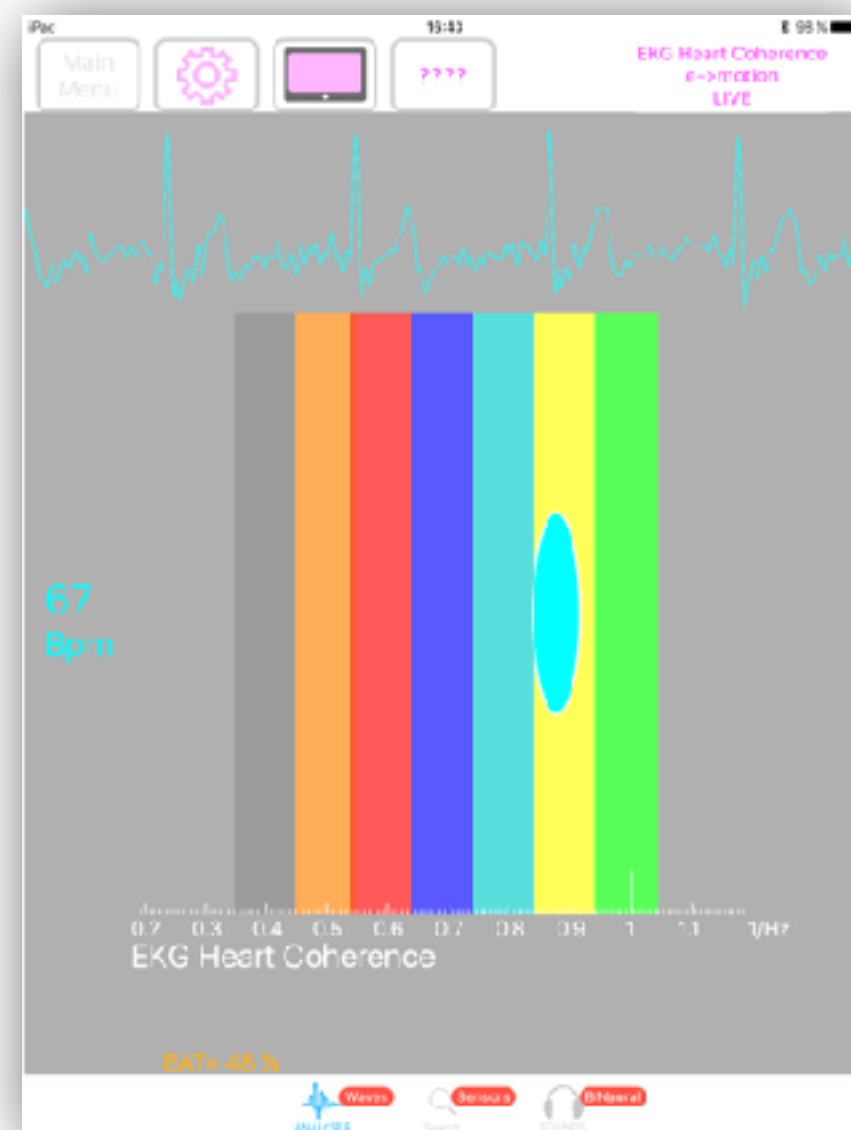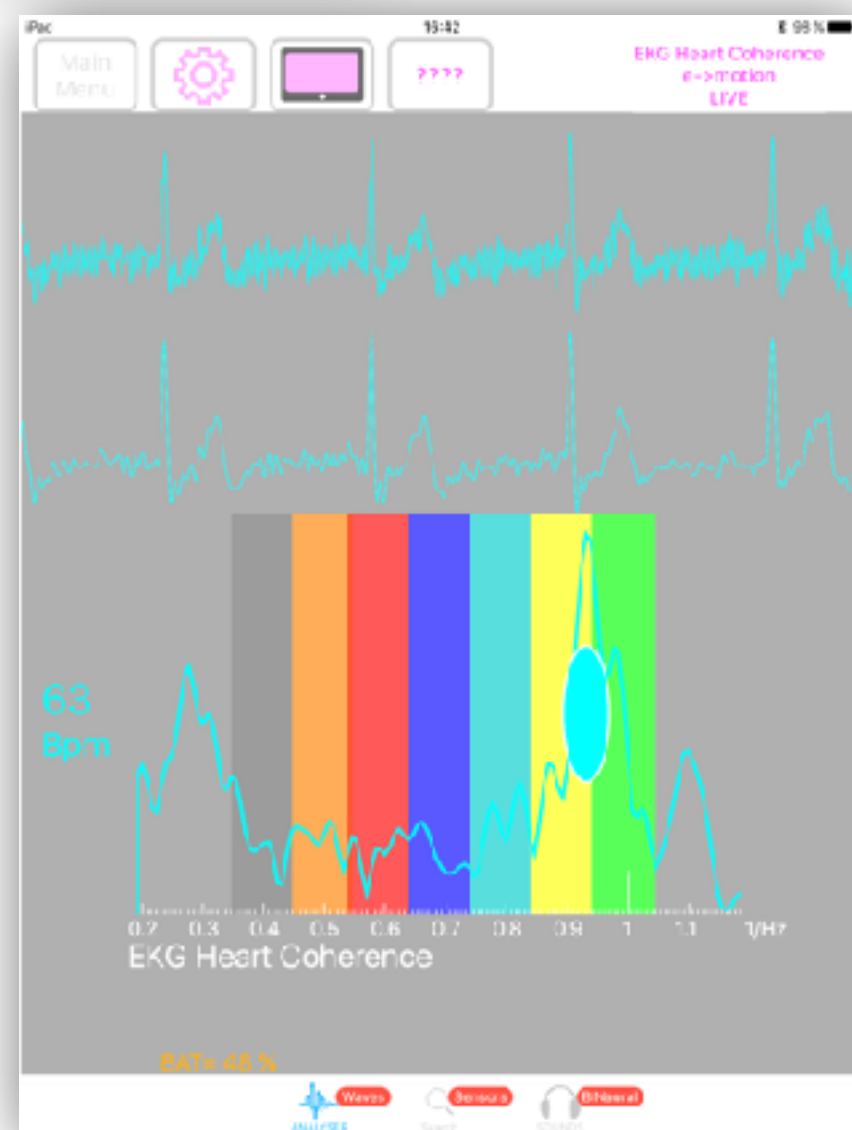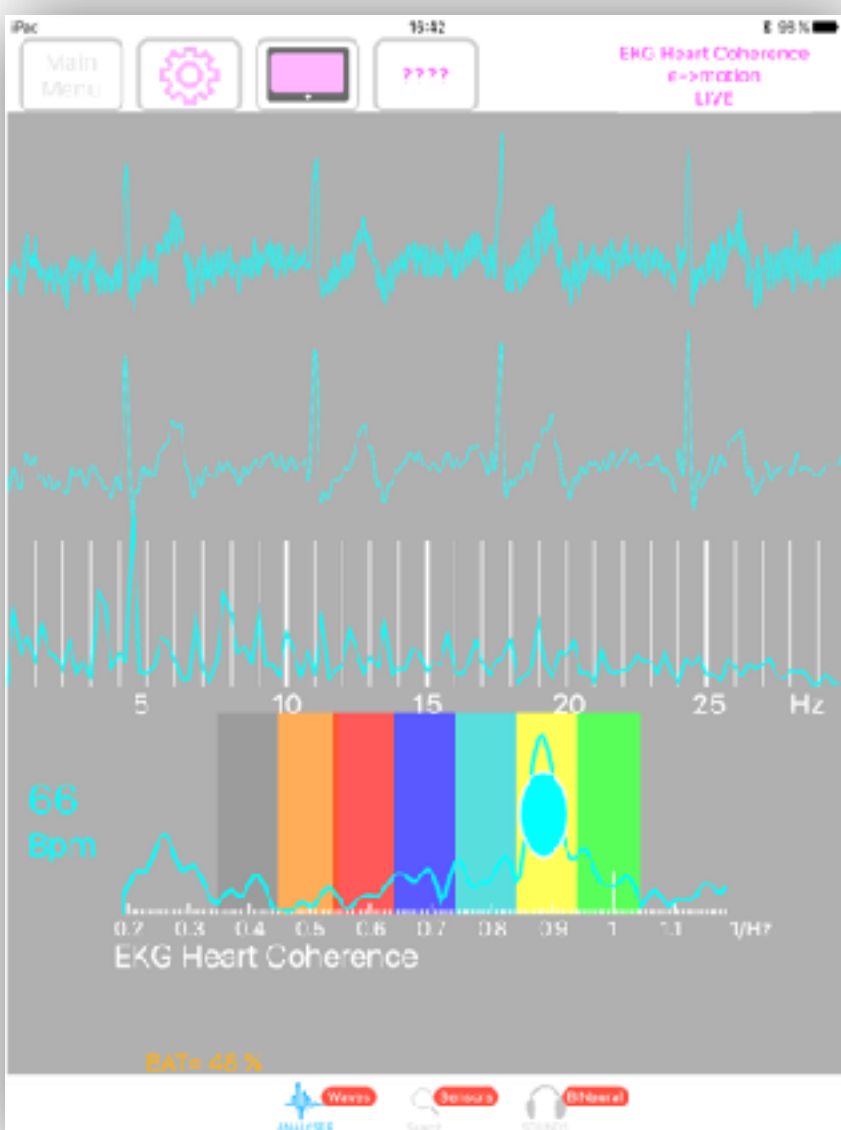


2 Users example

**Real Heart Coherence Measure from FlameinMind.com
Dan Winter & Patrick Botte - Here using Dan Winter's
original discovery - using CEPSTRUM- 2nd order power
spectra to measure INTERNAL coherence in EKG**


e->
motion
Graph

e->
motion
Spectrum

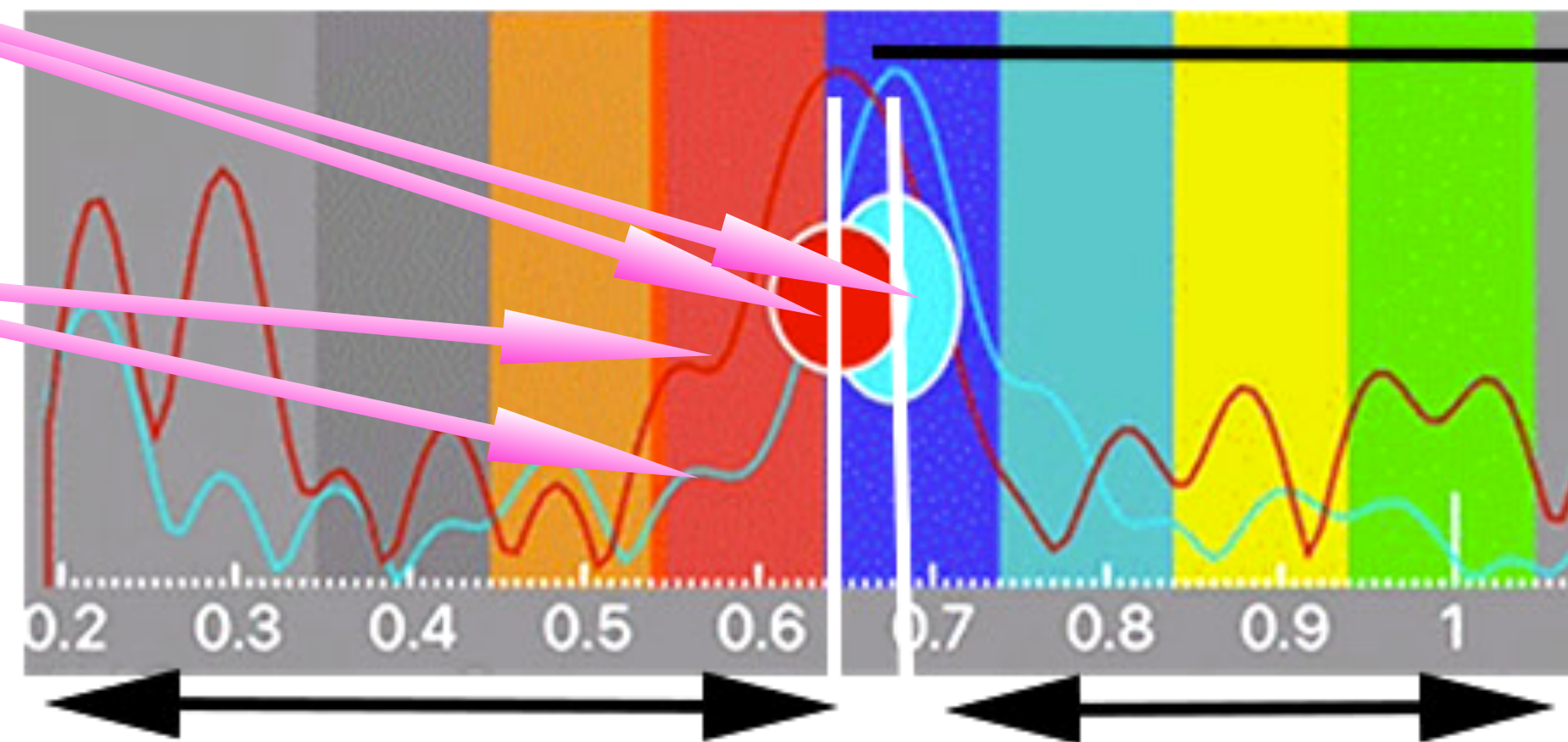

**The vertical AMPLITUDE
of this peaks measures
AMOUNT of Heart Coherence
(here also indicated for 2 persons
in SIZE of the blue/red egg shape)
-experience shows clearly
this 'measurement of EKG
becoming LASER like-
is a true measure of passion,
a LIE DETECTOR-
and functionally measures
the KINESIOLOGY
(body truth) response!**

**The moving right/left position of this cepstrum peak-
indicates the musical key signature of heart rate,
when 2 Hearts Line up- the physics of empathy/entrainment are measureable- (even Tantra)
also experience shows peaks near .61 are more TOUCH PERMISSIVE /'heart centered'
versus peaks nearer to 1.0 are invariable more CONCEPTUAL /'head centered' emotion.**

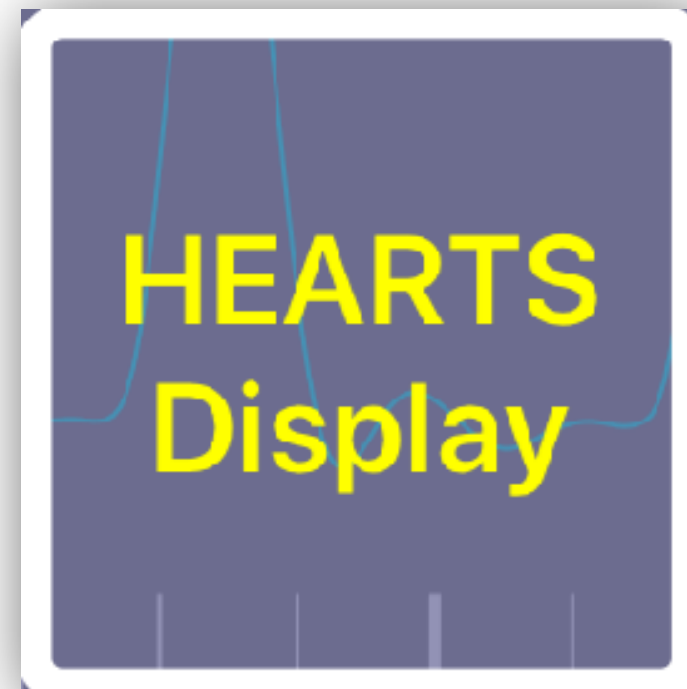

Hearts Display (2 Users)

The left heart is channel 1
The right heart is channel 2

The size of each heart is related to its internal coherence.
The relative positions of the hearts are relative positions of the emotional peaks (musical key signature) related to the spectrum analysis


Positions of the hearts are NOT related to spectrum scale displayed with

e->
motion
Graph

OR

e->
motion
Spectrum

If only **HEARTS Display** option is selected only hearts & Bpm are displayed

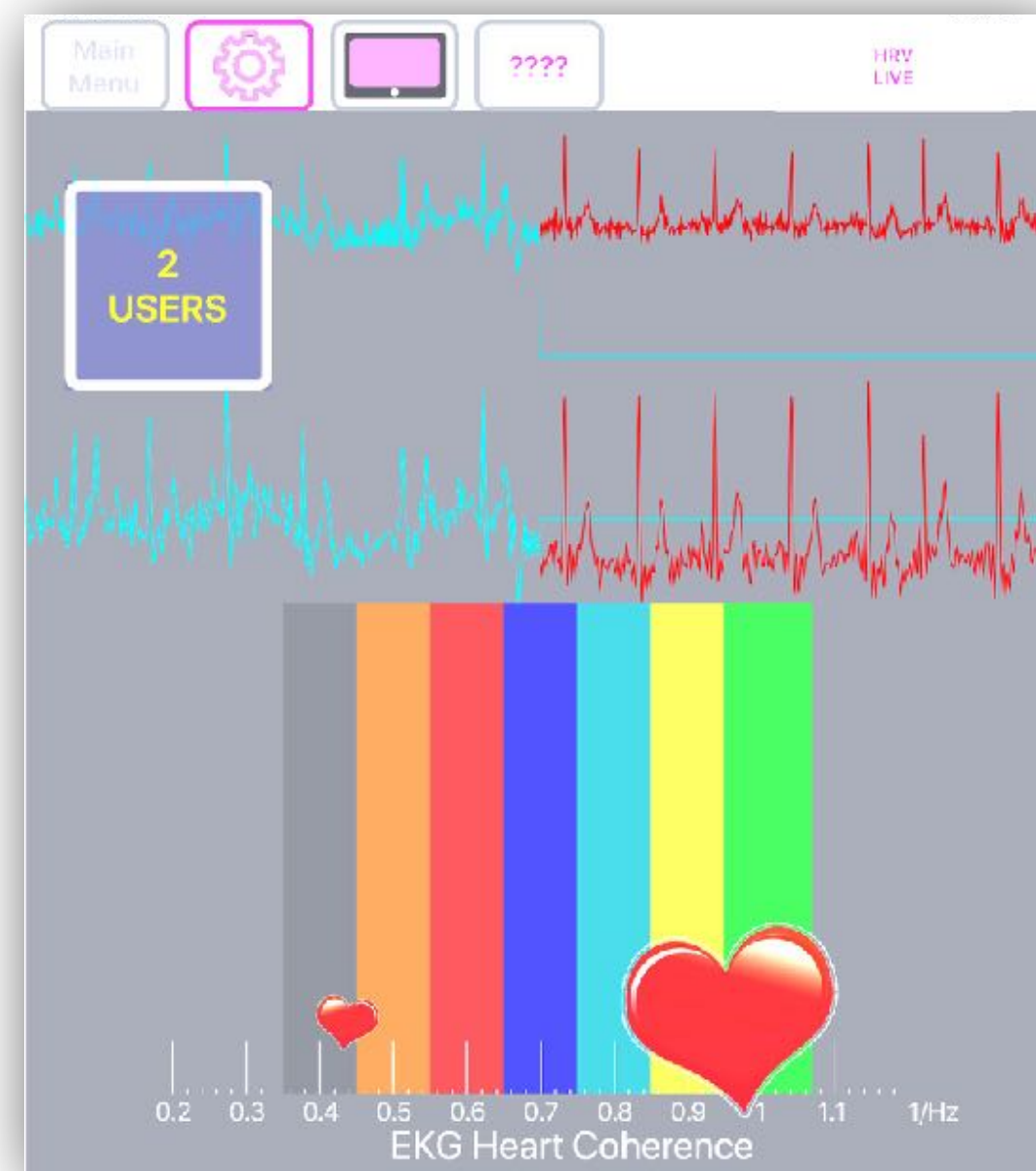

51
Bpm

76
Bpm



81
Bpm

63
Bpm



76
Bpm

63
Bpm



76
Bpm

75
Bpm

